

GP300 Operation Manual

Software Installation

For first time use of the Blood Glucose Meter converter, you must install the drive for the USB device. You can find the driver on the disk included in the PC Link System package.

Converter Installation



Before you connect the converter, remove the Smart code Key from the meter.



When the Converter is connected, you will see a connection message (PcL) appear on the screen of the meter.



Insert the smaller adapter of the USB cable into the Converter.




Insert the bigger adapter of the USB cable into your USB port.

Using the Software


To be able to do a thorough statistical analysis of your blood glucose data, you need DIABASS Blood Glucose Management software. You can download the software from <http://www.diabass.info/>. The Bionime PC Link System package also includes an optional purchase of DIABASS.

Start Download

To start downloading the saved data from the meter to the computer, you need to connect the meter and the computer with a USB cable. Start up DIABASS software and click on Import.

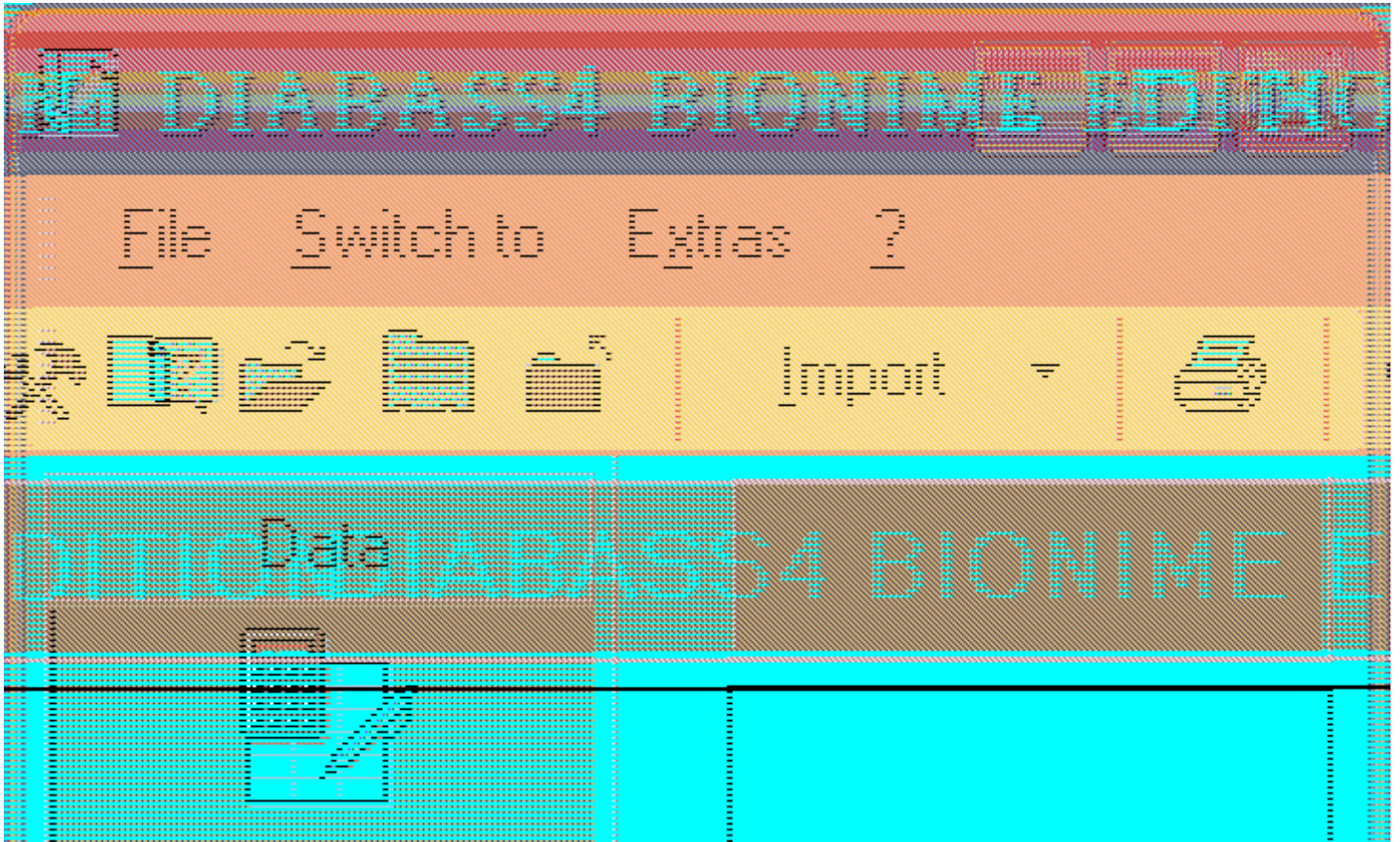

Choose your BIONIME meter and follow the instructions to upload the data.

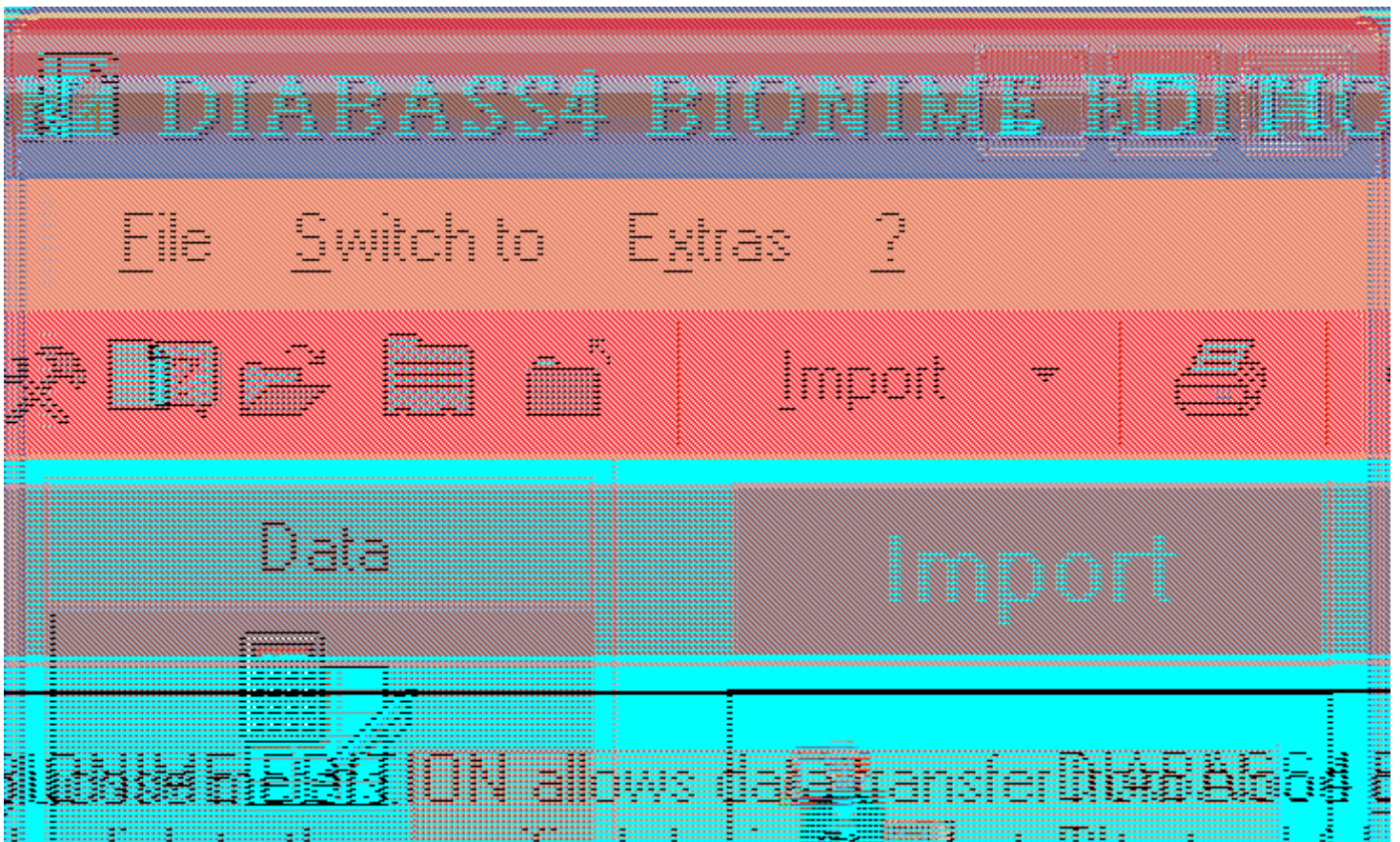

For more detailed information on how to use DIABASS, please refer to the user manual.